

Bed Rail Protectors

SET - UP AND OPERATION

These super soft, foam Bed Rail Protectors are designed to protect patients from injury through limbs striking or getting caught in the metal bed rails.




- ⚠ Please read these instructions before use
- ⚠ Do not allow sharp objects to penetrate the cover
- ⚠ This medical device should only be used in conjunction with correct medical advice and under the supervision of a suitably qualified healthcare professional

To attach protectors

To attach the protector to your bed rail, release the hook and loop fastening strip at the base of the protector. Place the protector over the bed rail, fastening strip at the bottom, and press strip together firmly to reseal. When using collapsible bed rails, check the bed rail protectors do not interfere with the operation of the rails.

Simply release the hook and fastening strip to remove. Do not fold protectors for storage, or place heavy objects on the surface when not in use.

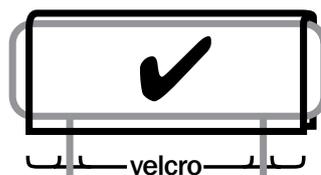

Regular maintenance

Regularly check the bed rail protectors are properly positioned and securely fastened to the bed rails.

Regularly check the bed rail protector for damage and wear, paying special attention to the condition of the fastening strip. Do not use the bed rail protectors if you see evidence of any rips, tears or other damage or general wear.

Care & Cleaning

Wipe surface with a mild soap and warm water solution and rinse well. Dry with a clean cloth. For heavier soiling, wipe clean with a diluted bleach solution and rinse well before drying. Do not use water temperatures in excess of 80°C.

May be air or tumble dried at low temperature. Remove immediately once dry. Do not iron, wring or twist.

Do not use harsh abrasives or phenol based cleaning agents. If dry cleaning, do not use a solution stronger than perchloroethane.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications

SIZE	Standard (Pair)	Large (Pair)
CODE	BL5247	BL5254
LENGTH	1260 mm	1580 mm
WIDTH	335 mm	490 mm
CONFORMS TO	ARTG176837	ARTG176837

MADE IN CHINA