


Swivel Bath Seat

SET UP AND OPERATION

The Swivel Bath Seat is manufactured from lightweight powder-coated metal with moulded plastic seat, contoured back rest and side support arms.

It comes fully assembled from the factory.

- ⚠ Please read these instructions before use.
- ⚠ This bath seat is designed as a temporary seating aid only. Do not use the seat as a step or transfer device.
- ⚠ Before use, ensure the rim of the bath is wide enough to support the seat frame.

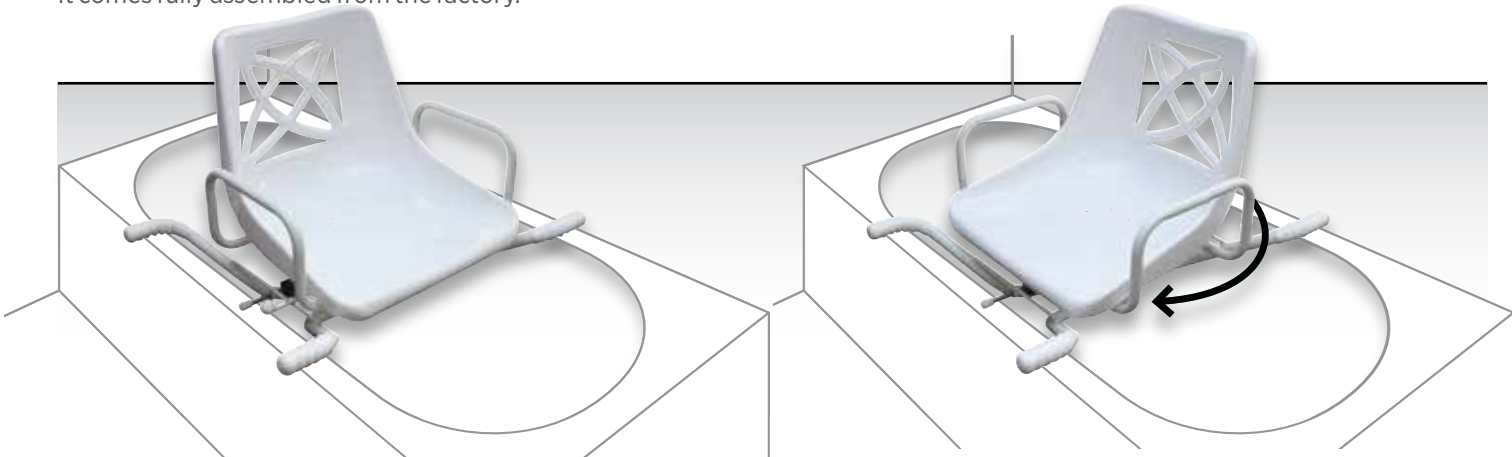

Positioning the seat

Place the bath seat over the bath, with self locking release lever fitted on either side of the bath as preferred. Each of the four non-slip handles should have a minimum of 25 mm resting on the edge of the bath.

Operating the seat

If bathing unassisted, ensure the user has sufficient strength to operate the swivel lever mechanism.

The seat can swivel a full 360° in either direction, locking at 90° intervals. The swivel mechanism is operated by pulling up on the lever to release the lock and allow the seat to turn. The lever may be stiff at first but will ease with use. The seat turns in 90° intervals before the lock reengages.

Swivel the seat to face the side of the bath ready for transfer. Always check how close the edge of the seat is in relation to the rim of the bath, as transfer assistance may be required if the seat is positioned too far from the edge of the bath. Ensure the seat is locked in position before the user is seated.

Unlock the seat by lifting the lever and swing the legs over the side of the bath while the seat is slowly turning 90°. The users legs can be lifted together or one at a time. Before bathing or showering, ensure the seat is securely locked in position.

For added comfort, it is recommended that towelling or other material be placed over the seat – especially for male users.



Adjust each positioning screw until the rubber tip presses firmly against the side of the bath. For easier transfer the seat can be set closer to the side of the bath by readjustment of the positioning screws.

Before use, ensure the bath seat is squarely and securely positioned over the bath.

Regular maintenance

Regularly check the bath seat and frame for damage and wear, paying special attention to the condition of the seat, backrest and non-slip handles.

Regularly check the function of the locking mechanism.

Do not use the bath seat if the plastic seat or backrest is cracked or deformed in any way, or if the locking mechanism is faulty.

Replace the handles immediately if you see evidence of rips, tears, cracks or other general wear.

Care & cleaning

⚠ When using the Swivel Bath Seat, do not use any bath salts or other corrosive additive in the bath water.

Doing so may cause corrosion which is not covered under the product warranty.

To clean the bath seat and frame after each use, wipe clean with a mild soap and warm water solution and rinse well. Dry with a clean cloth.

For heavier soiling, wipe clean with a diluted bleach solution and rinse well before drying. Do not use water temperatures in excess of 80°C.

Avoid harsh, abrasive cleaning agents or utensils (such as a wire brush) as these can lead to deterioration of the frame and seat.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months.

Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or manufacturer for a list of available spare parts.

Specifications

CODE	BL1522
COLOUR	White
MAXIMUM USER WEIGHT	110 kg
PRODUCT WEIGHT	6 kg
OVERALL WIDTH	730 mm
SEAT WIDTH	430 mm
OVERALL DEPTH	450 mm
SEAT DEPTH	430 mm
OVERALL HEIGHT	500 mm
SEAT HEIGHT	355 mm
WARRANTY	1 year
CONFORMS TO	ARTG 176831 AS 4069-2013
MADE IN CHINA	