

Toilet Surround

SET - UP AND OPERATION

The Toilet Surround is manufactured from powder-coated steel. The height adjustable surround is designed to fit most toilets, with moulded anti-slip armrests for comfort, and anti-slip rubber tips for safety.

It comes fully assembled from the factory.




- ⚠ Please read these instructions before use
- ⚠ Users must be able to push with equal force on both armrests so as not to tip the surround

Adjusting the height

The surround is height adjustable to match individual user requirements. A series of holes along each leg allow leg height to be adjusted from 680 mm to 810 mm.

⚠ Ideal armrest height

With the user seated, ideal armrest height is at elbow height, with user's arms resting against their sides and lower arms resting flat on armrest. Upper arms should never flare outwards during transfer.

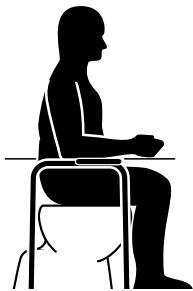


A brass locking button is used to secure the height of each leg.

Press in the locking button to disengage, and turn the leg slightly so the button is no longer aligned with the holes. Move the leg up or down to your ideal height setting, then turn the leg back to align the locking button with the nearest hole.

The locking button must be fully engaged and protruding through the height adjustment hole to ensure a positive lock.

Repeat for all four legs, adjusting height evenly or individually to match an uneven floor surface.

Before use, test that locks are fully engaged and ensure the surround is stable and resting firmly on the floor's surface.

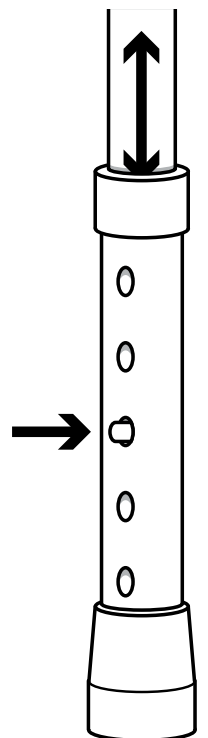

Toilet Surround

Regular maintenance

The leg locking buttons should be regularly inspected to ensure they are fully engaged.

Regularly check the toilet surround for damage and wear, paying special attention to the condition of the armrests and rubber tips. Do not use the surround if the armrests are cracked or deformed in any way. Replace the rubber tips immediately if you see evidence of rips, tears, cracks or other general wear.

Care & cleaning

It is recommended the surround be cleaned after heavy use or at least once a week. To clean, wipe with a soft clean cleanser and warm water solution and rinse well. Dry with a clean cloth.

For heavier soiling, wipe clean with a diluted bleach solution and rinse well before drying. Do not use water temperatures in excess of 80°C.

Avoid harsh, abrasive cleaning agents or utensils (such as a wire brush) as these can lead to deterioration of the surround.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to enclosed Warranty Statement for full details.

Specifications

CODE	BL1165
COLOUR	White
CAPACITY	190 kg
PRODUCT WEIGHT	4.3 kg
OVERALL WIDTH	580 mm
OVERALL DEPTH	440 mm
OVERALL HEIGHT	680 > 810 mm
BETWEEN ARMRESTS	480 mm
WARRANTY	1 year
CONFORMS TO	ARTG 176831
	ISO 11199-1:1999
	Walking aids manipulated by both arms
MADE IN CHINA	