

## Walking Stick Holder

### SET UP AND OPERATION

Designed to attach to the frame of a wheeled walker for easy storage and transportation of a walking stick.

Can be attached to either side of the frame as required.

Suitable for all BetterLiving wheeled walker models excluding the Tri-Wheel.

CLIP




Prior to use please read all instructions

Suitable for most 'A' framed wheeled walkers with a tubular frame diameter of 25 mm and handle diameter of 23 mm

Suitable for most walking sticks to a maximum diameter of 23 mm

Do not attempt to place anything other than a walking stick into the holder

CUP

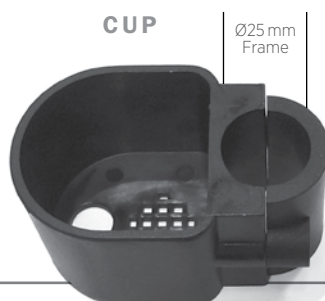

#### Attaching the upper clip

The upper clip must be mounted to the handle tube of the wheeled walker (on either side as preferred). To allow sufficient space, the handle cannot be set on the lowest height setting.

Unscrew the 2 bolts holding the upper clip bracket using the hex key provided. With the clip facing outwards, place the brackets around the handle tube on your preferred side. Reinsert both bolts and tighten.

Note that depending on handle tube size, bracket may not close together completely however it must be sit firmly and securely around the frame without movement up or down.



#### To attach the lower cup

The lower cup should be mounted just below the side brace of the wheeled walker where possible, or on a lower section of frame that does not interfere with the operation of the side brace.

Unscrew the 2 bolts holding the lower cup bracket using the hex key provided. With the cup facing outwards, place the brackets around the frame (below the side brace, where possible) on your preferred side. Reinsert both bolts and tighten.

The cup must be mounted in a straight line to the clip.

Typically, both upper lower attachments sit perpendicular to the frame however they may need to be angled to suit the design of the individual wheeled walker.

Ensure both upper clip and lower cup attachments are securely attached to the frame before use.


## To insert a walking stick

Place the base of your walking stick into the lower cup, then push the walking stick against the upper clip until it clicks into place (with walking stick handle facing outwards).

Pull gently to release.

Ensure the handle of the walking stick does not obstruct the handles or interfere with the operation of the brakes.

Always ensure the walking stick is securely attached before using the wheeled walker.

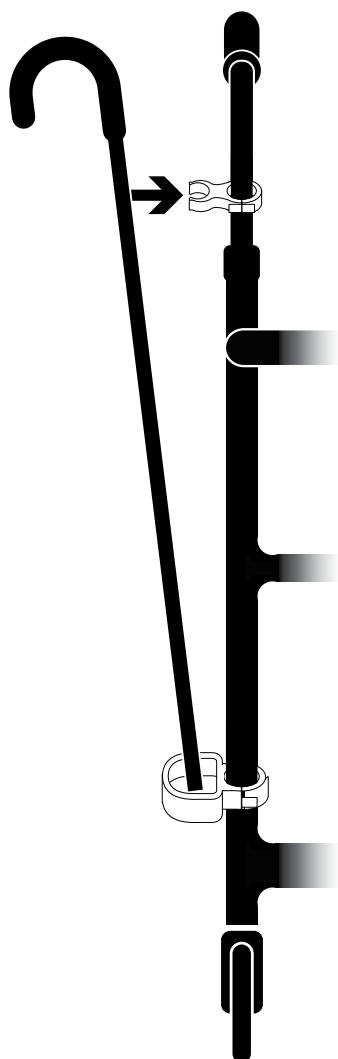

## Care & cleaning

To clean the walking stick holder, wipe with a clean damp cloth and dry thoroughly. Avoid using abrasive cleaning agents or utensils (such as a wire brush).

## Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

## Specifications

CODE	BL1478
MAXIMUM DIAMETER	23 mm (maximum stick width)
WEIGHT	0.15 kg
CONFORMS TO	ARTG 176830
	Made in China